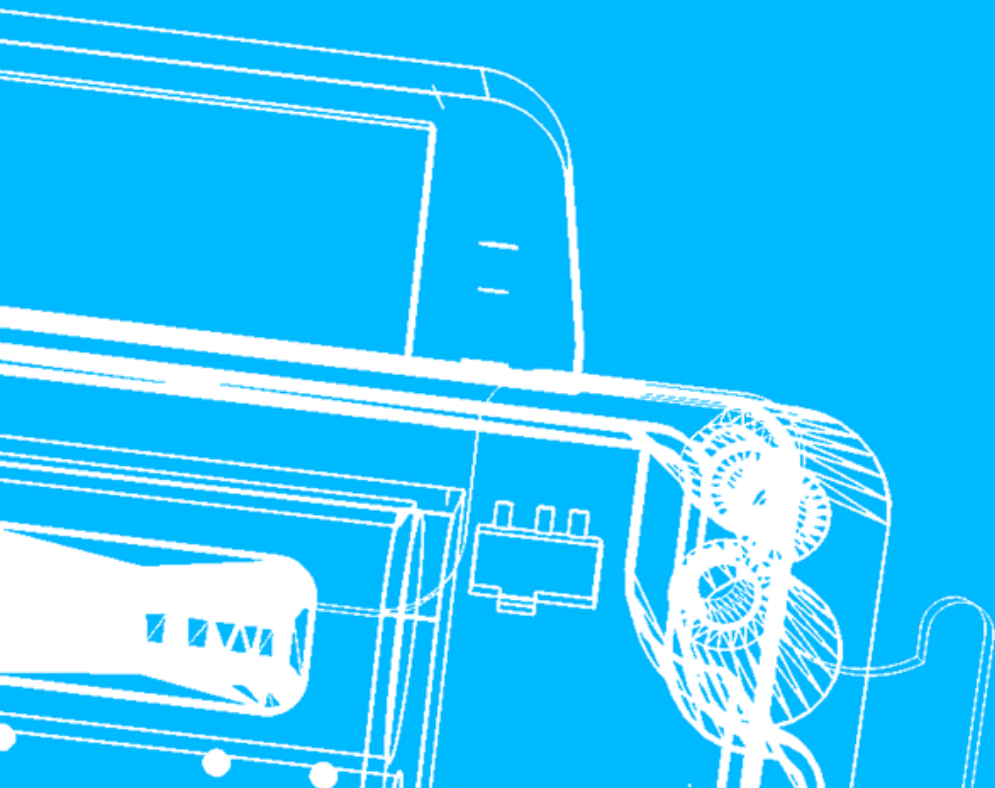
BIRTHSENSOR

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User manual

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1 Birthsensor in een oogopslag.....	3
2 Aan de slag.....	4
3 Veiligheid, gebruik en ondersteuning.....	10
4 Probleem oplossen.....	11
1 Birthsensor at a glance.....	12
2 Getting started.....	13
3 Safety, use and support.....	19
4 Troubleshooting.....	20
Specificaties, Specifications.....	21

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Overzicht van de Birthsensor

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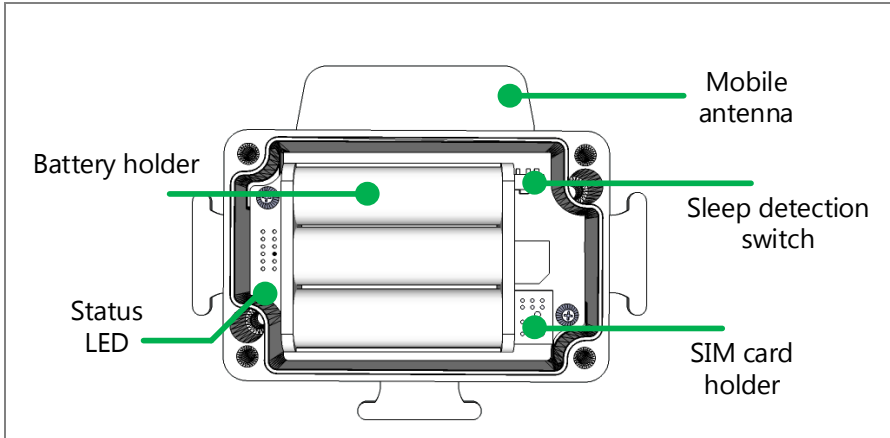
Birthsensor at a glance

Birthsensor overview

This section describes the various components of the open birthsensor.

Sleep detection switch:

Used to enable or disable sleep detection mode.
Further information can be found on page 15.

SIM card holder:

Place the SIM card here.

Battery holder:

Place 3 AA batteries here.

LED:

The LED shows the status of the birthsensor.
Page 17 provides an overview of the various statuses.

Antenna:

An external antenna that provides better mobile coverage than an integrated antenna.

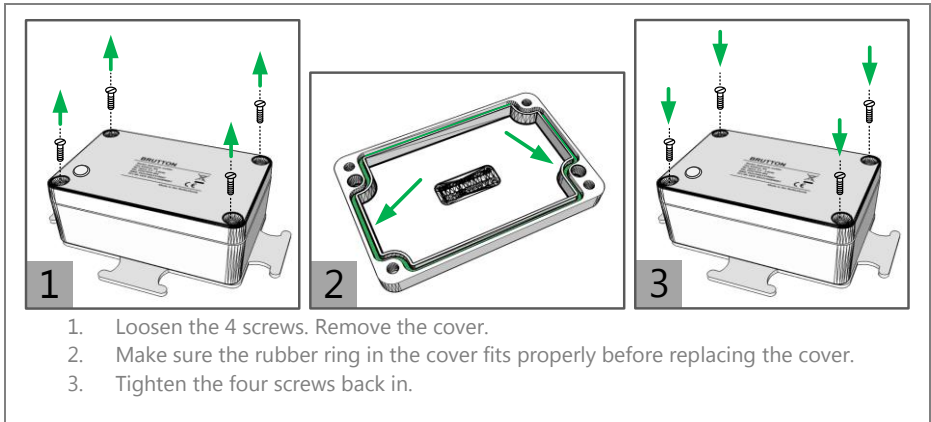
Getting started

Warning: To avoid injuries, read the safety information in section 3 before using Birthsensor.

Opening and closing Birthsensor

You need to open the birthsensor before adjusting the settings and placing the batteries. Follow the instructions below to open the birthsensor.

Tip: always use a suitable screwdriver to avoid damaging the screws.

Entering telephone numbers directly

Telephone numbers can be added to the directory of a SIM card easily by placing it in a telephone. Follow the steps detailed in the instruction manual of your telephone. Give the registered numbers the names "TEL1", "TEL2" and/or "TEL3". Use upper-case letters only. See the example below. Other combinations are possible.

Attention: make sure that the SIM card only contains the telephone numbers for the birthsensor.

Example 1:

Name	Number
TEL1	062222222
TEL2	+3162222222
TEL3	041322222

Example 2:

Name	Number
TEL3	041322222

Tip: The telephone numbers can also be added to the directory of a SIM card by sending a SMS to the Birthsensor.

Placing the SIM card

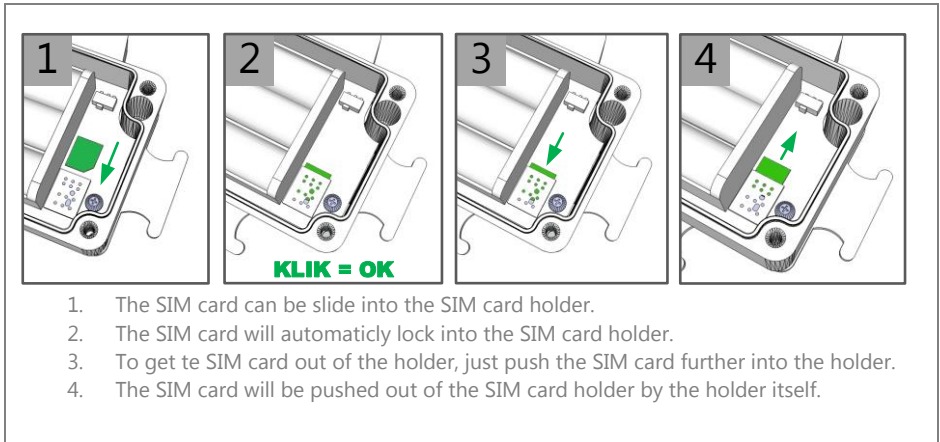
Attention: Remove the batteries before placing the SIM card.

Attention: Make sure that the following SIM settings have been adjusted before placing the SIM card in its holder.

- Check that the SIM card's PIN code is "0000".
- Check that the SIM card has enough call credit.

The best way of carrying out these checks is by putting the SIM card in a telephone.

Follow the instructions below to place the SIM card follow.



The birthsensor's status LED flashes green if it fails to detect a SIM card. The birthsensor's status LED flashes red if it fails to detect telephone numbers in a SIM card.

Entering telephone numbers via SMS

You can also add numbers to the SIM card easily via SMS. The birthsensor will delete all existing numbers from the SIM card and replace them with those in the text message.

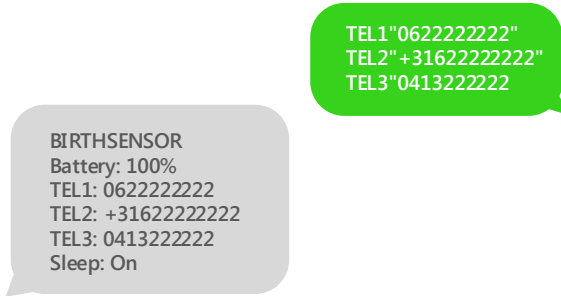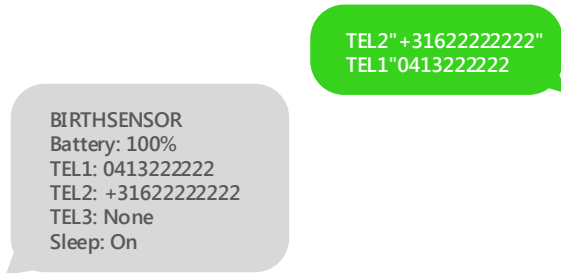Start the text message with the name of telephone numbers TEL1, TEL2 and/or TEL3 followed by the telephone numbers between double quotation marks. Use upper-case letters only for the names. You can enter the numbers in the text message in any order see the examples below.

Example 1 SMS:**Example 2 SMS:**

The status LED remains orange while telephone numbers are being entered. It can take a couple of seconds for the text message to be received. The birthsensor will restart after the telephone numbers have been entered. After startup the Birthsensor will send a status SMS back. The telephone numbers stored in the SIM card are showed in the Status SMS.

Sleep detection switch

Mares sometimes lay down to sleep, which can result in the birthsensor issuing a false alarm. The sleep detection mode should be engaged in such cases:

TIP: Since the sleep detection mode delays notification, it should be disabled unless strictly necessary.

Sleep detection off: Sleep detection is disabled, and an alarm will be issued if the mare lays down for over 8 seconds.

Sleep detection on: If sleep detection is enabled, an alarm will not be issued immediately after 8 seconds. A notification will go out only if the mare changes its position within 3 minutes. This happens when the mare tries to catch its breath after a contraction. The birthsensor will go into sleep mode if the mare does not change its position within 3 minutes. This happens when the mare falls asleep. The birthsensor exits sleep mode as soon as further movement is detected.

Placing batteries

The batteries can be placed after inserting the SIM card and adjusting the sleep detection switch. NiMH batteries with a charge of at least 2,400mA/h are recommended. Birthsensor also works with alkaline high-energy batteries (for photo cameras). Regular alkaline batteries should not be used because they do not provide enough energy. Always replace the 3 batteries at the same time and use the same type of batteries.

Tip: Thanks to its ultra-low power consumption, Birthsensor is the only birthsensor on the market that can also be used with rechargeable batteries. This saves lots of non-rechargeable batteries and protects the environment.

Place the batteries in the specified position in the battery holder.

The right batteries will power the birthsensor for about a month.

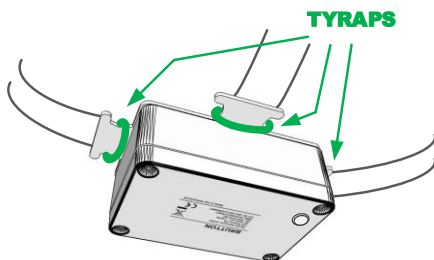
Tip: Always check that the batteries are charged before setting up the birthsensor.

Attention: Placing the batteries in the wrong position could damage the sensitive electronics of Birthsensor. Remove the batteries if you are not going to use the birthsensor for long periods in order to prevent damage to the birthsensor from battery leakage.


Mounting at a halter

The Birthsensor can be mounted under a halter by three tyraps. Make sure that the Birthsensor is placed in the middle below the head of the mare. When the Birthsensor is not placed in the middle of the halter it can lead to no or false alarms.

Using Birthsensor

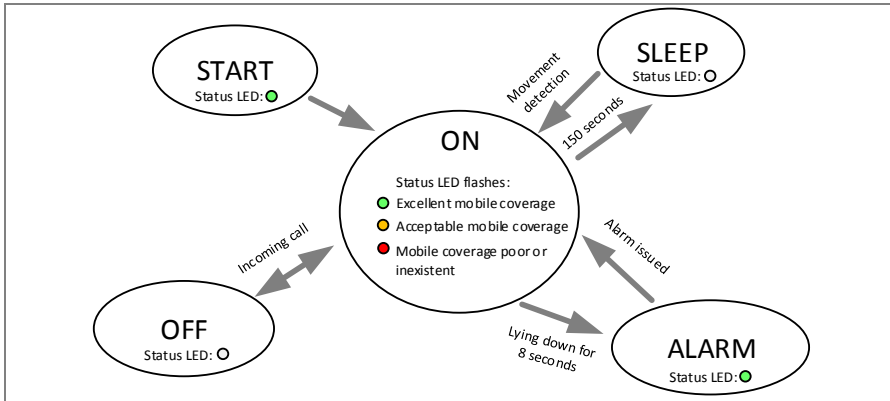The following diagram explains how the birthsensor works. The birthsensor will turn on automatically (START) after the batteries are placed. This may take a couple of seconds. The status LED remains green throughout the START phase.

The status LED will start flashing green fast after turning on the device if the birthsensor fails to recognise the SIM card. Make sure that the SIM card is adjusted properly. The status LED will start flashing red fast after turning on the device if the birthsensor fails to find any numbers on the SIM card. Make sure that the SIM card is adjusted properly. The status LED will start flashing orange fast after turning on the device if battery levels are too low. Replace the batteries.

The birthsensor will enter ON mode if all adjustments have been performed correctly. The status LED will flash red, orange or green every five seconds depending on the mobile coverage.

- Red = mobile coverage poor or inexistent
- Orange = acceptable mobile coverage
- Green = excellent mobile coverage

The birthsensor will enter SLEEP mode after 150 seconds to save power. The status LED will remain off. The birthsensor switches from SLEEP mode to ON mode whenever it detects movement.

In ON mode, the birthsensor checks the status of the mare every 0.25 seconds. This algorithm reduces the likelihood of false alarms and is much more precise than the switch elements used in other alarms.

Whenever the mare lays down for 8 seconds (sleep detection off) or it lays down for 8 seconds and then stands back up within 3 minutes (sleep detection on), the device will call the telephone number listed under "TEL1" (ALARM). Do not answer the call. Wait for it to end. You can then call the birthsensor back within a minute. Wait until you have heard at least one tone. The other numbers will be called if you do not call back within a minute.

TIP: Make sure your voice mail is always off. Your voice mail answers calls automatically.

You can disable the birthsensor with the back of the screwdriver (magnet). Swipe the back of the screw driver on the side of the enclosure where the status led is located. The status LED will turn red when the birthsensor is turning off. Swipe again and the birthsensor will turn on (green).

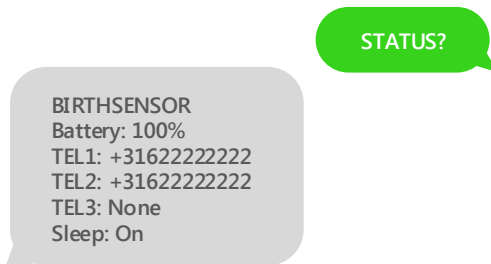
TIP: Always check that the alarm works before setting it. This should be done at the location where the mare is kept so that you can test the mobile coverage. You can try the device by tilting the birthsensor 90 degrees for over 8 seconds.

Text message: "Battery level low!"

The birthsensor sends a text message whenever battery levels fall below 10%. This message is only sent to the first number to be called in the event of an alarm. In most cases, this will be "TEL1", failing which the message will be sent to "TEL2" and so on. Batteries should be replaced as soon as possible. The other numbers receive no text messages.

Status SMS

The birthsensor can send a status text message to give information about the current status. The status text message will be send when the birthsensor receives a text message with "STATUS?" inside. The birthsensor will send a status text message back.

Safety, use and support

Important safety information

Warning: Failing to follow the safety instructions below may lead to fire, electric shocks and other injuries, as well as damage to Birthsensor or other property. Therefore, read the following safety information before using Birthsensor.

Handle the electronics of the birthsensor carefully. Keep the interior free of dust and liquids.

Repairs: do not attempt to repair the birthsensor on your own. Disassembly by non-experts may damage the delicate electronics.

Batteries: use only the specified batteries for the birthsensor. Using other types of batteries may lead to overheating, injuries or damage to the birthsensor or other property. Remove empty batteries from the birthsensor. Read the batteries' safety instructions.

Radio signals: the birthsensor uses radio signals to access the mobile network. These electromagnetic waves can interfere with medical devices (pacemakers).

If you have a medical device, keep it always at a distance of 30 cm or more from the birthsensor. If you suspect that the birthsensor is interfering with the functioning of your medical device, stop using the alarm and see your doctor.

Temperature: The birthsensor is designed for use at temperatures of -10 to 50 °C and can be stored at temperatures of -40 to 75 °C. Using the birthsensor outside this temperature range may lead to damage.

Troubleshooting

Birthsensor flashes red/green/orange when turned on

Problem: Flashing green: the birthsensor has failed to detect a SIM card.

Solution: Check that the PIN code and/or telephone numbers have been properly set.

Problem: Flashing red: the birthsensor has failed to find TEL1, TEL2 or TEL3 in the SIM directory.

Solution: Check that the PIN code and/or telephone numbers have been properly set.

Problem: Flashing orange: battery levels are too low.

Solution: Replace the batteries.

Birthsensor cannot be turned on

Problem: Battery levels may be too low.

Solution: Replace the batteries.

Birthsensor works but fails to make calls

Problem: The SIM card may have no call credit left.

Solution: Check and/or increase your call credit.

Problem: The birthsensor may have poor mobile coverage.

Solution: Check the mobile coverage.

Birthsensor calls are too short or stop abruptly

Problem: The battery type may not be the correct one.

Solution: Place NiMH or alkaline high-energy batteries (for photo cameras). Birthsensor's power requirements mean it cannot work with normal alkaline batteries.

Birthsensor calls the registered numbers repeatedly

Problem: Your voice mail may be enabled.

Solution: Disable your voice mail following your provider's instructions.

Problem: The call is answered.

Solution: Do not answer the call but wait for it to end and then call the Birthsensor back.

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Warranty

Birthsensor is covered by the 2-year warranty period mandated by the EU. The Brutton warranty does not cover defects arising from normal wear, accidents or misuse of the product. Brutton declines any liability for damages arising from use of the birthsensor.